
Chemfort™ Closed System Transfer Device (CSTD) Reconstitution and Administration Components

www.simplivia.com

3961444 VER.2023-12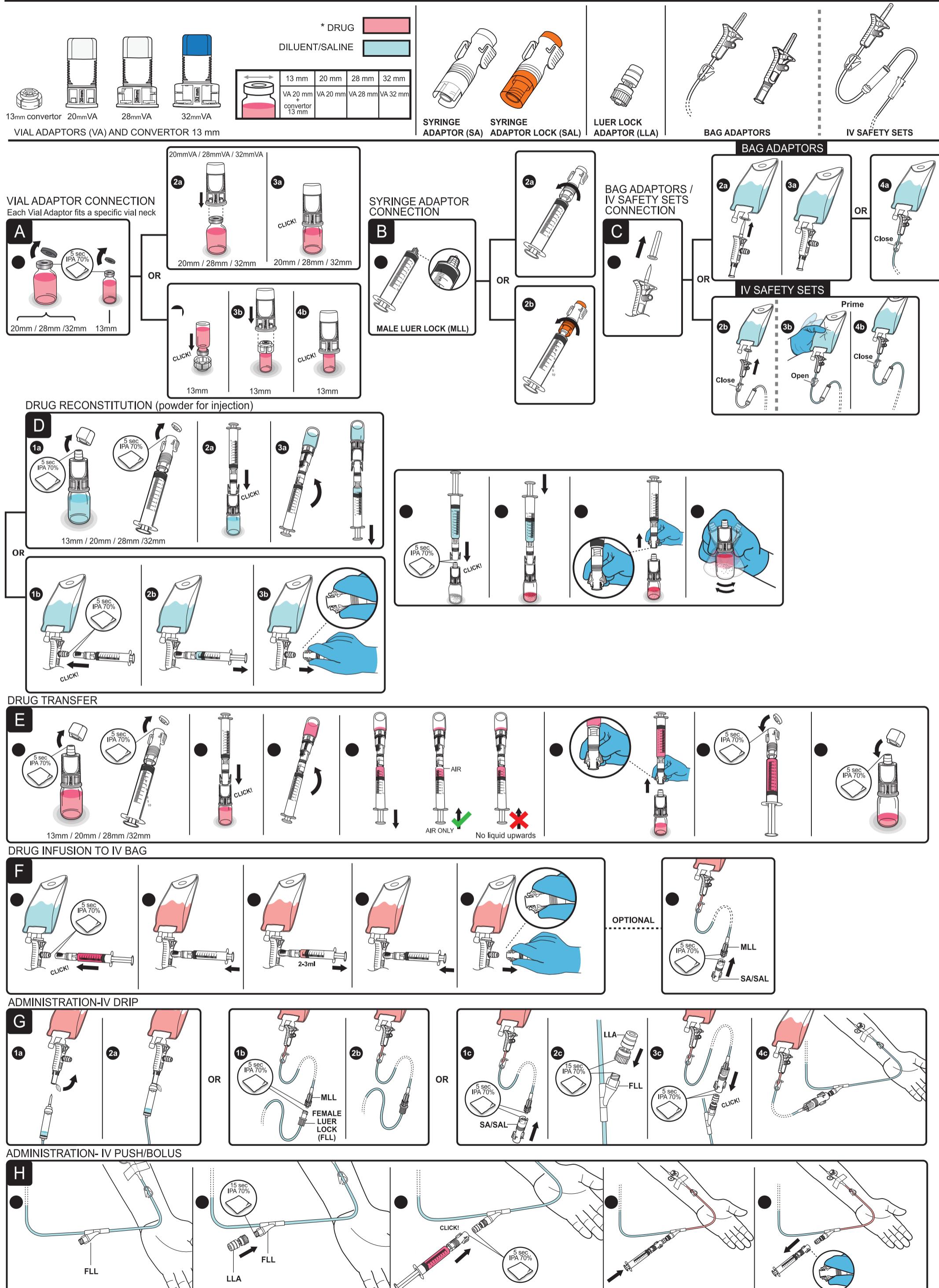

GB

INDICATIONS FOR USE

Chemfort™ is a Closed System Transfer Device (CSTD) that mechanically prohibits the release of drugs, including antineoplastic and hazardous drugs, in vapor, aerosol or liquid form during preparation, reconstitution, compounding and administration, minimizing exposure of individuals, healthcare personnel, and the environment to hazardous drugs.

Chemfort™ prevents the introduction of microbial and airborne contaminants into the drug or fluid path for up to 7 days.

NOTES

1. **Chemfort™** has been shown to prevent microbial ingress for 7 days and 10 activations.
2. Replace tubing sets per European Centre for Disease Prevention and Control guidelines/ local regulation and institutional protocol.
3. Disposal of components should be according to local regulations and procedures for handling of hazardous drugs.
4. Refer to local distributors for additional training information.
5. Not made with natural rubber latex or DEHP.

PRECAUTIONS

1. Use Aseptic technique.
2. Do not disconnect **Chemfort™** devices from other non **Chemfort™** components.
3. **Chemfort™** can be used with the drug up to 7 days.

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Explanation of symbols taken from ISO 15223-1:2021

ref: 5.1.1		Manufacturer	ref: 5.3.2		Keep away from sunlight	
ref: 5.1.2		Authorized representative in the European Community / European Union	ref: 5.3.4		Keep dry	
ref: 5.1.4		Use-by date	ref: 5.4.2		Do not re-use	
ref: 5.1.5		Batch code	ref: 5.4.3		Consult instructions for use	
ref: 5.1.6		Catalog number	ref: 5.6.3		Non-pyrogenic	
ref: 5.2.3		Sterilized using ethylene oxide	ref: 5.6.4		Drops per millilitre	
ref: 5.2.6		Do not resterilize	ref: 5.6.5		Liquid filter with pore size	
ref: 5.2.8		Do not use if package damaged	ref: 5.7.7		Medical device	
ref: 5.2.11		Single sterile barrier system	ref: 5.7.8		Translation	
ref: 5.2.14		Single sterile barrier system with protective packaging outside		CE 0483		
* DRUG	GB Medicine Diluent/saline	SE Medicin Utsprädningsmedel/saltlösning	CZ Lék Redidlo / fyziologický roztok	NO Medisin Fortynningsmeddel/saliner	GR Ιατρική Διατύπωση/αλετουχο	RS Lijek Razredivač/sol
DILUENT/SALINE	DE Medikamente Diluent/salin	PT Medicamento Solvente/soro fisiológico	LV Medikaments Atšķaidītājs/fizioloģiskais šķidums	FI Lääke Läämennin/suolaliuos	MT Medicina Dilwent/salina	HB הירוח המלח דוד
	FR Médicament Diluant/solution saline	ES Medicamento Diluyente/solución salina	DK Medicin Oplosning/saltlösning	PL Medycyna Rozcieńczalnik/sól fizjologiczna	RO Medicament Solvant/solutie salină	
	NL Medicijn Diluent/saline	SK Lek Rozpúšťadlo/fyziologický roztok	EE Ravim Lahendusvedelik/ Füsioloogiline lahush	BG Лекарство Разтворител/физиологичен разтвор	SL Zdravilo Topilo/fiziološka raztopina	
	IT Medicinale Diluente/soluzione salina	HU Gyógyszer Hígítószer/sóoldat	LT Vaistas Skiediklis / druskos tirpalas	HR Lijek Razredivač/fiziološka otopina	IR Leigheas Caolaitheoir/saline	