## TEVADAPTOR® Drug Reconstitution and Transfer System with TOXI-GUARD® filter

- DE** TEVADAPTOR® System zur Rekonstitution und zum Transfer von Arzneistoffen - mit TOXI-GUARD® - Filter
- FR** TEVADAPTOR® Système de reconstitution et de transfert de médicament avec filtre TOXI-GUARD®
- NL** TEVADAPTOR® systeem met TOXI-GUARD® - filter voor bereiding en toediening van (intraveneuze) geneesmiddelen.
- IT** TEVADAPTOR® Sistema per ricostituzione e somministrazione di farmaci con filtro TOXI-GUARD®
- SE** TEVADAPTOR® System för läkemedelsberedning och administrering med TOXI-GUARD®-filter
- PT** TEVADAPTOR® Sistema de reconstituição do medicamento e de transferência com TOXI-GUARD®
- ES** TEVADAPTOR® Sistema de reconstitución de fármaco y transferencia con filtro TOXI-GUARD®
- SK** TEVADAPTOR® Systém na prípravu a prenos pomocou filtra TOXI-GUARD®
- HU** TEVADAPTOR® Gyógyszer-elkészítési és átviteli rendszer TOXI-GUARD® szűrővel
- CZ** TEVADAPTOR® Systém na přípravu a přenos léku pomocí filtru TOXI-GUARD®
- LV** TEVADAPTOR® Zāļu šķīdināšanas un pārneses sistēma ar TOXI-GUARD® filtru.
- DK** TEVADAPTOR® System til blanding og administrering af lægemidler med TOXI-GUARD® filter
- EE** TEVADAPTOR® ravimi kokkusegamise ja ülekandmise süsteem koos TOXI-GUARD® filtriga
- LT** TEVADAPTOR® Vaistų atskiedimo ir perpylimo sistema su TOXI-GUARD® filtru
- NO** TEVADAPTOR® rekondisjonering og overføring av legemiddel med TOXI-GUARD® filter
- FI** TEVADAPTOR® lääkkeiden sekoitus- ja siirtojärjestelmä TOXI-GUARD®-suodattimella
- PL** TEVADAPTOR® z filtrem TOXI-GUARD® System do rekonstytucji i podania leku
- HB** TEVADAPTOR® מערכת למיהול והעברה של תרופות עם TOXI-GUARD® פילטר

 **Medical Ltd.**

**North Industrial Zone**

**P.O. Box 888, Kiryat Shmona, Israel**

**Zip Code: 1101801**

EC REP

Donawa Lifescience Consulting Srl  
Piazza Albania, 10  
00153 Rome, Italy

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3961643 VER. 2018-10

GB

## TEVADAPTOR® Drug Reconstitution and Transfer System with TOXI-GUARD® filter

### INSTRUCTIONS FOR USE

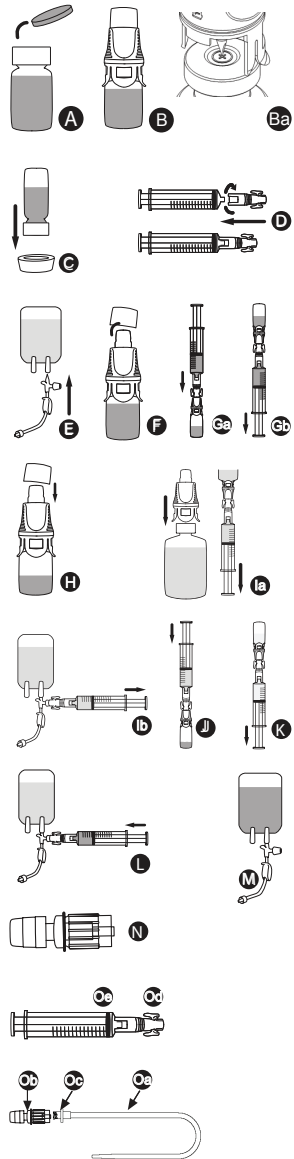
1. Remove protective cap from vial. (A)
2. Keep the vial on a flat surface (B), insert the spike of the Vial Adaptor in the center of the rubber stopper, keeping the spike perpendicular to the stopper surface. (Ba)  
For 13 mm vials, use the Vial Converter first. Place the 13 mm Vial converter on a flat surface, inverting the vial and pushing into Converter (C), then connect the 20 mm Vial Adaptor. For 20 mm vials, use 20 mm Vial Adaptor.  
For 28 mm vials, use 28 mm Vial Adaptor.  
For 32mm vials, use 32mm Vial Adaptor.
3. Connect Syringe Adaptor to a luer lock syringe. (D)
4. Spike Connecting Set/Spike Port Adaptor/ Infusion Set into an infusion bag spike port, prime Connecting Set / Infusion Set and clamp, or clamp Spike Port Adaptor. (E)  
(Step 4 not applicable for syringe compounding).

#### 5.1 For Drug Transfer (drug solution)

- a) Remove Vial Adaptor protective cap. (F)
- b) Connect Syringe Adaptor to Vial Adaptor (Ga) invert and withdraw desired drug volume. (Gb)  
(If necessary, remove air from syringe by gently pushing plunger in upside down orientation, until resistance is first felt. Do not push hard, as membrane may be plugged).
- c) Gently disconnect Syringe Adaptor from Vial Adaptor by pinching wings and allowing Syringe Adaptor spring to slowly push components apart. (Partial vial: Recap Vial Adaptor and store vial for further use). (H)

#### 5.2 For Reconstitution (powder for injection) and Drug Transfer

- a) Remove Vial Adaptor protective cap. (F)
  - b) Draw desired amount of diluent from a vial (i) or bag (ii)
    - i. Click a new Vial Adaptor onto diluent vial, connect Syringe Adaptor and draw diluent from vial (Ia), or
    - ii. Connect Syringe Adaptor to Connecting Set / Spike Port Adaptor / Infusion Set and draw desired amount of diluent from bag. (Ib)
  - c) Gently disconnect Syringe Adaptor
    - i. from Vial Adaptor, or
    - ii. from Connecting Set / Spike Port Adaptor / Infusion Set by pinching wings and allowing Syringe Adaptor spring to slowly push components apart.
  - d) Connect Syringe Adaptor to Vial Adaptor on drug vial, inject diluent and dissolve drug. (J)
  - e) Invert and withdraw desired volume from vial into syringe. (K)  
(If necessary, remove air from syringe by gently pushing plunger in upside down orientation, until resistance is first felt. Do not push hard, as membrane may be plugged).
  - f) Gently disconnect Syringe Adaptor from Vial Adaptor (See 5.2.c)  
Partial vial: recap Vial Adaptor and place the Vial Adaptor & the Vial in a perpendicular position only, for further use. (H)
6. Connect Syringe Adaptor to Connecting Set / Spike Port Adaptor / Infusion Set and inject drug into the bag. (L)  
Rinse spike and Syringe Adaptor by withdrawing 2-3ml of solution from bag and reinjecting.
  7. Gently disconnect Syringe Adaptor from Connecting Set / Spike Port Adaptor / Infusion Set. (See 5.2.c)

#### Symbol legend

8. a) Deliver bag with reconstituted drug to hospital ward. (M)  
b) Alternate mode (Syringe Compounding)  
(Step 4 not applicable).  
After step 5, cap Syringe Adaptor and deliver to ward for push injection.
9. For push / bolus injection, Connect Luer Lock Adaptor (N) to female luer connector (e.g. stopcock) on infusion set. Close line toward IV cannula.  
Prime stopcock & Luer Lock Adaptor by connecting an empty syringe with Syringe Adaptor and with drawing 1-2 ml solution. Disconnect Syringe Adaptor from Luer Lock Adaptor and close IV line upstream of injection port.  
Administer drug by connecting drug filled syringe with Syringe Adaptor to Luer Lock Adaptor and injecting drug.

#### WARNINGS

- Do not fill vial in an inverted position.
- In repetitive use of the Vial, the Vial Adaptor & the Vial should be placed in a perpendicular position only.
- The Syringe Adaptor is designed for use with luer lock syringes.
- If Syringe Adaptor is being reused, do not disconnect from syringe and use only with the same drug.
- Do not use with 13mm vials of LYOPHILIZED drug.
- Do not remove Flow Stop from Connecting Set or Infusion Set, unless when connecting to delivery set or patient.
- Do not remove Vial Adaptor from the vial
- Do not use with drug containing N, N-Dimethylacetamide (Such as amsacrin or busulfan)
- The Vial Adaptor may be used up to 7 days with most drugs  
Use of the Vial Adaptor is limited to 72 hours with drugs that contain a combination of the following ingredients:
  - 1) Polysorbate 80 and Ethanol
  - 2) Ethanol and Cremophor such as: Macrogol glycerol ricinoleate / Poloxyl 35 Castor Oil  
(For example: Etoposide and paclitaxel)
- Disposal of components should be according to local regulations and procedures for handling of hazardous drugs.
- Individual component septums may be pierced up to ten times.
- Reuse of single-use devices creates a potential risk for patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
- The Tevadaptor system contains the following materials: ABS, Polyisopren, Polycarbonate, PVC-DEHP-free.
- Before using the Tevadaptor system refer to the Drug Information Leaflet in order to verify that there are no restrictions regarding the plastic components used in the Tevadaptor system.
- For Syringe Adaptor Lock:  
Once the syringe is connected to Syringe Adaptor Lock, it cannot be disconnected

#### System components:

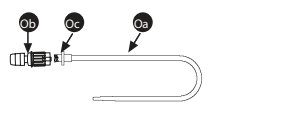
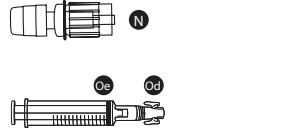
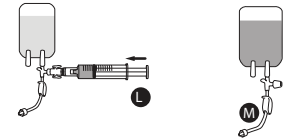
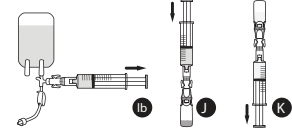
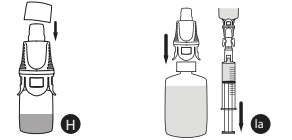
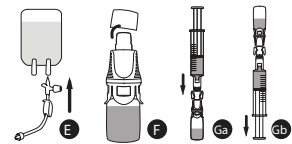
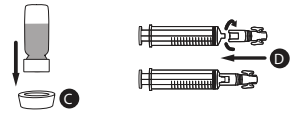
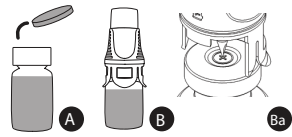
- 13mm Vial converter
- 20mm Vial Adaptor
- 28mm Vial Adaptor
- 32mm Vial Adaptor
- Syringe Adaptor
- Syringe Adaptor Lock
- Connecting set (with flow stop cap)
- Spike Port Adaptor
- Infusion Set (with flow stop cap)
- Luer Lock Adaptor

#### For Bladder Instillation Procedure

##### Notes:

1. Use luer lock urinary catheter. (Oc)  
After insertion of the catheter (Oa) into the urethra, connect the Tevadaptor Luer Lock Adaptor (Ob) to the catheter's luer lock inlet. (Oc)
2. Attach the Tevadaptor Syringe Adaptor (Od) to the syringe (Oe) and the Tevadaptor Vial Adaptor to the drug vial.  
Withdraw the required amount of drug after clicking both the assembled units together.
3. Disconnect the Tevadaptor Syringe Adaptor from the Vial Adaptor and click the syringe - Syringe Adaptor onto the Tevadaptor Luer Lock Adaptor.
4. Deliver the drug into the bladder.
5. Gently disconnect Syringe Adaptor from Luer Lock Adaptor.
6. Upon treatment completion remove catheter without disconnecting Luer Lock Adaptor.

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