

Prevention of Hazardous Drug Vapor Release by the Tevadaptor® Vial Adaptor



Figure 1.

0.2 μm membrane

Activated charcoal filter

TOXI-GUARD®

Summary

Tevadaptor® is a Closed System Drug Transfer Device (CSTD) designed to prevent the escape of hazardous drug vapors into the environment during drug reconstitution and administration¹. Drug containment in the Tevadaptor® Vial Adaptor is accomplished by the TOXI-GUARD® system, which contains a 100% activated carbon² drug binding matrix and 0.2 μm hydrophobic and oleophobic membrane (Figure 1). The activated carbon matrix is highly efficient in adsorption of drug vapors. The 0.2 μm membrane is a sterile barrier preventing microorganisms and particles from entering the system and, due to its hydrophobic and oleophobic properties, preventing aerosols and liquids from being released from the system. Together, they serve as an effective sterile, particulate and toxic drug vapor barrier.


The TOXI-GUARD® System ensures that the Tevadaptor® air pathway only allows particulate and bacteria-free air to enter the drug vial during drug reconstitution and preparation. It also ensures that the air that exit the drug vials is free of vapor of hazardous medications.

Studies were performed, challenging the efficacy of the Tevadaptor® to prevent the escape of drug vapors^{3,4}. A model system was designed to induce drug vapors within the drug vial. Since under normal usage conditions, the drug vapors that are generated are minimal, extreme conditions were employed to significantly increase vapor quantity. Vapors released from the Tevadaptor® were trapped within a closed test chamber. The trapped drug was collected and then analyzed by highly


sensitive LC/MS/MS methods.

The Tevadaptor® was challenged with four commonly used antineoplastic drugs: Carboplatin, Cyclophosphamide, 5-Fluorouracil and Paclitaxel (Figure 2). With Carboplatin, Cyclophosphamide and 5-Fluorouracil, vapors were consistently detected in control samples in which the TOXI-GUARD® system had been removed from the Tevadaptor®. In test samples containing an intact TOXI-GUARD® system, **no drug vapors were detected**. With Paclitaxel no drug vapors were detected in either the positive control or test sample. These results support the validity of the Tevadaptor® Vial Adaptor to prevent release of hazardous drug vapors.


Figure 2. Chemical structures of the anti-neoplastic drugs used for challenging the Tevadaptor® Vial Adaptor and the TOXI-GUARD® System. Figure shows the wide range of the drugs tested in terms of their size (molecular weight) and structural complexity.

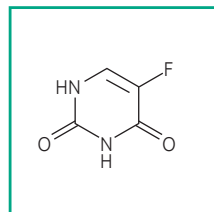


5-Fluorouracil
C₄H₃FN₂O₂
Daltons 130

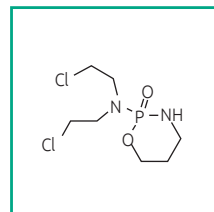

Cyclophosphamide
C₇H₁₅Cl₂N₂O₂P
Daltons 261

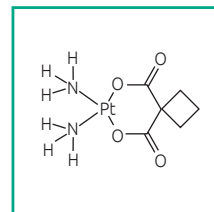

Carboplatin
C₆H₁₂N₂O₄Pt⁻²
Daltons 371

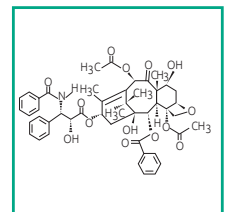

Paclitaxel
C₄₇H₅₁NO₁₄
Daltons 854


TDPT-IL-00103

Introduction


The Tevadaptor® Vial Adaptor is a closed system transfer device component that equalizes the pressure inside the vial without any need of action or activity by the user, thereby saving time and preventing potential errors¹. The Vial Adaptor spike contains two channels. One channel serves as the air pathway (see Figure 3, item 3) and the second channel as the liquid pathway (see Figure 3, item 4). The TOXI-GUARD® System has a sterile 0.2 µm hydrophobic and oleophobic membrane (Figure 3, item 2) on the interior side of the air channel

Figure 3. Left. Cross-cut of the Tevadaptor® Vial Adaptor. (1) TOXI-GUARD® active carbon matrix, (2) 0.2 µm hydrophobic and oleophobic membrane, (3) air path and (4) liquid path.

Figure 3. Right. Activated Carbon Cloth Matrix (Flexzorb™) in the TOXI-GUARD® system. Top panel, unmagnified picture. Bottom panel, magnified picture of the active carbon cloth, showing the tight weave of the carbon cloth matrix.

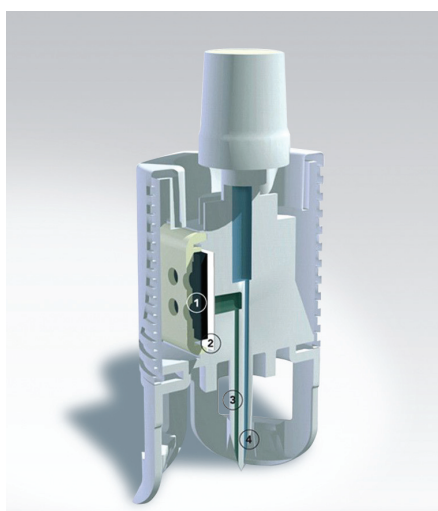

and a 100% activated carbon drug binding matrix on its exterior side (Figure 2, item 1). The hydrophobic membrane blocks passage of aqueous liquids out of the air channel, while maintaining high air permeability. The manufacturing process for the activated carbon matrix results in a woven carbon cloth with a highly microporous structure and strong electrostatic forces (Figure 4). This matrix is highly efficient at adsorbing active molecules that may pass through the 0.2 µm filter, preventing their release into the environment².

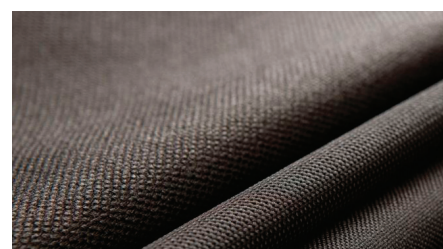

About Flexzorb™

Protection against toxic gases is one of the oldest applications of activated carbon, dating back to its use in World War I for protection against chlorine and other gases. Today it is used for a variety of industrial, military and medical applications. This includes removal of toxic and volatile gases in chemical manufacturing plants, in water purification systems, in industrial and military respirators, as protective clothing against chemical, biological or nuclear agents, and as wound dressings for protection against microbial infection.

The Flexzorb™ 100% activated carbon cloth is manufactured by Chemviron Carbon. Their special manufacturing process results in the

cloth having a micro-porous matrix, with an extremely large surface area (1000-2000 m²/g). The surface area of one gram of activated carbon cloth is over half the size of a football pitch. The microporous structure and cloth weave, gives the cloth a very high air permeability. The large surface area of the cloth, combined with the strong electrostatic forces that are induced in the cloth as part of the manufacturing process, and its high air permeability, results in the carbon cloth having very rapid adsorption kinetics. In addition the active carbon cloth is suitable for use in applications where there is a high humidity as its adsorption capacity is less adversely affected by moisture².

Test Method

The efficacy of TOXI-GUARD® system to prevent release of hazardous drug vapors was evaluated by employing a closed test chamber for capture of released drug vapors. Since the quantity of drug vapors that may be generated under normal use conditions is extremely low, and typically below analytical limits of detection, a model system was developed using extreme laboratory conditions to induce and generate drug vapors to a much larger extent than what would be found in typical working environment in hospitals and pharmacies. This entailed heating the drug vial and its solution to elevated temperatures (50-60°C) and having a constant stream of nitrogen gas flow into the vial via the Tevadaptor®

Vial Adaptor fluid pathway. Vapors released from the Tevadaptor® were trapped and then recovered by dissolving in the appropriate diluent. LC/MS/MS methods developed and validated specifically for each test drug, were employed to detect and quantify the amount of drug recovered. In order to verify that the test conditions resulted in drug vaporization, parallel testing was performed using Tevadaptor® Vial Adaptors in which the TOXI-GUARD® system had been removed (positive control). For each drug tested, the quantity of drug recovered from the sealed test chamber when intact Vial Adaptors were challenged was compared to the quantity of drug recovered in the Positive Control sample.

Test Results

Study parameters and results are listed in Table 1. Testing was performed at two reference laboratories, Analyst (Ness Ziona, Israel) and Nextar (Ness Ziona, Israel). The limit of quantitation (LOQ) in the LC/MS/MS systems ranged between 0.05-1 ng/ml, which represents a LOQ of 0.5-10 ng of recovered drug after compensating for the volume of diluent used to recover drug from the closed vapor trap chamber (10 ml). Representative chromatograms are shown in Figures 4. Drug vaporization at Analyst was performed using 90 L nitrogen gas at a 50°C drug incubation temperature and


at Nextar using 60 L nitrogen gas and 60°C temperature. With Cyclophosphamide, Carboplatin and 5-Fluorouracil, drug was consistently recovered in the positive control samples which had Tevadaptor® Vial Adaptors without the TOXI-GUARD® system, **not found in the test samples which had Tevadaptor® Vial Adaptors with the TOXI-GUARD® System.** With Paclitaxel, even under the extreme conditions that were employed, no drug was recovered in either the positive control or test sample.

Table 1. Quantity of Drug Recovered following Vaporization

Drug Tested	System LOQ ¹	Liters N2 Gas ²	Quantity Drug Recovered from Outside of the Vial Adaptor	
			Positive Control (TOXI-GUARD® Removed)	Test Sample (TOXI-GUARD® Present)
Cyclophosphamide ³	10 ng	90	32 ng	Below LOQ ⁵
Cyclophosphamide ⁴	0.5 ng	60	14 ng	Below LOQ
Carboplatin ³	10 ng	90	53 ng	Below LOQ
5 Fluorouracil ⁴	10 ng	60	147 ng	Below LOQ
Paclitaxel ⁴	0.8 ng	60	Below LOQ	Below LOQ

1 10 fold the LC/MS/MS Limit of Quantitation
 2 Liters of nitrogen gas used to induce the drug vapors
 3 Analyst Research Laboratories, Ltd. Rehovot, Israel Reference reports 2007-001 Et001C
 4 Nextar Chempharma Solutions, Ltd. Rehovot, Israel Reference reports 5560140RE-02, 5560140RE-03 and 5560140RE-04
 5 LOQ- Limit of Quantitation

Figure 4. Representative LC/MS/MS chromatograms of 5-Fluorouracil (top panel) and Cyclophosphamide (bottom panel). Left panels represent drug vapors recovered when the Tevadaptor® Vial Adaptor TOXI-GUARD® system is removed and the right panels represent drug vapors recovered when an intact Tevadaptor® Vial Adaptor TOXI-GUARD® system is present. Note that the Intensity Scale differs for the Positive Control Samples and the Test Samples. Magnification of the the Test Sample scale **28 fold** over the Positive Control Sample was performed in order to determine if a peak representing recovered drug could be detected over the background noise. **No peak was detected.** A similar procedure was followed for the Cyclophosphamide. **Even with the 17 fold amplification, no peak representing the drug could be detected over the background noise.**

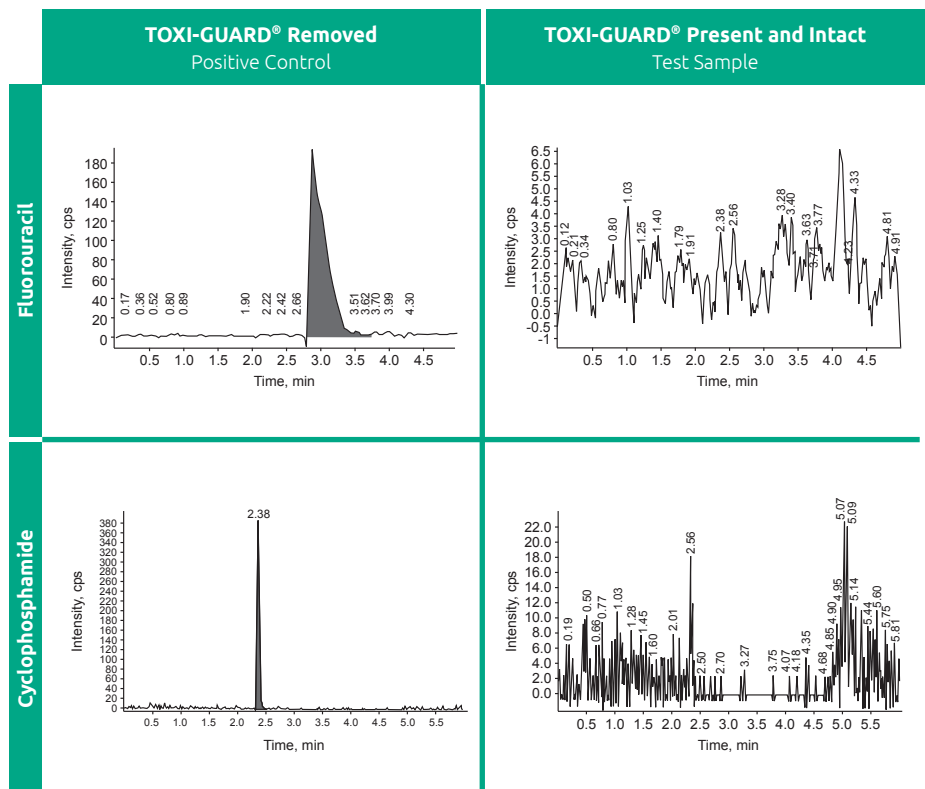

Study Conclusions

Extreme conditions were employed to challenge the efficacy of the Tevadaptor® Vial Adaptor's TOXI-GUARD® system to trap hazardous drug vapors. Four different anti-neoplastic drugs were utilized in the study. These drugs differ in size, physical properties and chemical formulation. With three of the four antineoplastic drugs tested, drug was recovered from the positive control samples in which the TOXI-GUARD® system was removed from the Tevadaptor® Vial Adaptor. Drug levels recovered in these Positive Control samples ranged between 14 ng to 147 ng. In contrast to these levels, in the Test Samples which had an intact Tevadaptor® Vial Adaptor TOXI-GUARD® system, drug levels were consistently below the level of quantitation. **The absence of recovered drug vapor in the test samples confirms the efficacy of the TOXI-GUARD® system present in the Tevadaptor® Vial Adaptor to stop hazardous drug vapors.**

With Paclitaxel no drug was detected in either the positive control samples or the test samples. This is most likely due to the excipients present in its formulation (for example Macroglycerol ricinoleate) and the overall viscosity of the solution. In contrast to the other drugs tested in this study, this formulation is not volatile, even under the extreme conditions used in the study.

The ability of the TOXI-GUARD® system to prevent vapor release with the different drugs that were tested, attests to the efficacy of the Tevadaptor® Vial Adaptor to meet the challenge of different drug structures and their formulations; whether it be a relatively small molecule such as 5-Fluorouracil or a large complex organic molecule such as carboplatin.

References

1. Tevadaptor Instructions for Use (IFU)
2. Flexzorb™ Activated Carbon Cloth Product Brochure published by Chemviron Carbon, Cloth Division, United Kingdom <http://www.chemvironcarbon.com>
3. Tevadaptor Data on file, Analyst Report, 2007
4. Tevadaptor Data on file, Nextar Report, 2014

